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Stand der Information: 2016-10

en FOR THE LUBRICATION OF CONTACT LENSES DURING WEAR

Instructions for use

VISLUBE® – Sodium hyaluronate from fermentation 0.18 %.

For the lubrication of contact lenses during wear.

Sterile, free from preservatives.

Composition: 1 ml solution contains sodium hyaluronate 1.8 mg, sodium chloride, potassium chloride, disodium phosphate, sodium citrate, magnesium chloride, calcium chloride and water for injections. The solution is hypotonic.

Indications: Sensation of dryness, discomfort or distorted vision during contact lens wear (rigid or soft).

Contra-Indications: Individual hypersensitivity to any constituent of the product.

Interactions: Do not use VISLUBE® at the same time as any drug or other product applied to the eye since it may modify their effects.

Undesirable effects: On very rare occasions transient disorders such as irritation of the conjunctiva, foreign body sensation, redness or burning sensation of the eye, short-term blurred vision may occur.

Dosage and administration: Twist off the tab. Tilt the head backwards and position the tip of the container above the eye to be treated. Move the lower eyelid down with the forefinger of one hand. Press the container and one drop of VISLUBE® will easily flow out (see illustrations). If not otherwise recommended by your contact lens specialist, instill 1–2 drops of VISLUBE® four times a day onto the contact lens during wear or onto the eye. After blinking, the solution will disperse and form a transparent and long lasting coating on the surface of the eye and lens.

Precautions: Do not touch the tip of the opened container and do not touch the surface of the eye with the tip of the opened container. Otherwise the container should be discarded immediately after use. Reclose the container after use. As VISLUBE® does not contain preservatives, the solution should be used within 12 hours after opening. Any solution not used within 12 hours after opening must be discarded. Otherwise the sterility is no longer guaranteed and this may be associated with a risk of infection. Do not use VISLUBE® if the container is damaged. Store below 30 °C! Do not freeze! Do not use VISLUBE® after the expiry date indicated on the container and the folding box. If discomfort persists while using VISLUBE® consult a physician. Keep out of reach of children!

Characteristics and mode of action: VISLUBE® contains sodium hyaluronate, a natural polymer, which is also present in the structures of the human eye. The particular physical characteristics of sodium hyaluronate confer to VISLUBE® its viscoelastic and water retaining properties. VISLUBE® provides a stable coating on the surface of the eye and lens

which is only slowly eliminated by blinking. Therefore VISLUBE® combines long lasting relief with maximum comfort. VISLUBE® is well tolerated due to its unique composition. VISLUBE® is free from preservatives.

Presentation: 20 single-dose containers of 0.3 ml. 60 single-dose containers of 0.3 ml.

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